



Cytotoxic / Chemotherapy Drug Spills

Cytotoxic Drug Spill Kit

Multiple use kit contains full Personal Protective Equipment and removal tools for safe containment and removal of cytotoxic drug spillages.

Cytotoxic Kit

Lightweight and portable for intensive and frequent use on multiple and/or large scale spillages. High Absorbency GV Health Cyto Granules absorb and contain liquid spills. Absorbent pads and sterile water mop up powder spills. Our custom Cytali solution designed to neutralise cytotoxic residue of spill site and surrounding area. Warning signs and colour coded, sealable bags for safe disposal.

Application

GV Health Spill Packs and Kits hold the NHS Supply Chain contract for cytotoxic spill cleaning. Used throughout the NHS in hospitals, clinics, surgeries, out-patient departments, pharmacies, laboratories; nursing and care homes and other locations where cytotoxic or chemotherapy drugs are handled and stored. Safe for use by non-specialist staff.

Contents

Protective Equipment

- 3 pairs of disposable gloves
- 3 thumb loop gown
- 3 FFP3 mask
- 3 eye visor
- 3 pairs of over sleeves
- 3 pairs of overshoes
- 1 kneeling pad

Liquid Clean up

- 3 Cyto Granule 50g sachet
- 3 scoop scraper
- 1 tweezers

Powder Clean up

- 3 sterile water pods
- 3 absorbent pads

Decontamination

- 3 Cytali Powder 10g sachet
- 1250ml diluter
- Paper towels

Waste disposal

- 1 instruction sheet
- 3 Cytotoxic waste bag
- 3 bag tie
- 1 warning sign
- 1 record card

Instructions / Procedures

Cytotoxic Spills

- 1** Put on all protective clothing: Gloves, Oversleeves, Overshoes, Face Mask, Visor & Apron/Gown.
- CHOOSE APPLICABLE STEP 2 FOR SPILL**

2 CYTOTOXIC LIQUID ON HARD SURFACES

Sprinkle **Cyto Absorbent Granules** over the spill & leave for 30 seconds. Use protective kneeling mat & brush off granules if broken glass or sharp material.

When the granules have absorbed the spill, cover waste bag and collect granules using the scoop & scraper and place in bag along with used scoop & scraper.

2 CYTOTOXIC LIQUID ON SOFT FURNISHINGS

Use the pad to absorb the spillage, placing on the spill and "blotting" the area, not "scrubbing". Dispose of pad in yellow bag when spill has been absorbed.

2 CYTOTOXIC POWDER ON HARD SURFACES & SOFT FURNISHINGS

Use water pods to dampen an absorbent pad. Place the pad gently over spill "blotting" not "scrubbing". For larger powder spills use scoop & scraper to scoop powder up into pad. Pad removed spill and pad in waste bag provided.
- 3** Pour **Cytali Powder** sachet into 200ml clear container. Fill container 3/4 full with Tap Water. Once powder has dissolved, use paper towels and solution to wipe surface where spill has been removed. Ensure area is fully wiped with solution and leave to dry.
- 4** Pour unused solution into sink with running water. Rinse out 200ml container so it is ready for reuse. Ensure all used protective clothing and items are put in waste bag provided. Gloves in bin. Tie bag top to seal. Wash hands thoroughly after.
- 5** Ensure bag and all contents are incinerated correctly.

GV Health Ltd, South Hill, Cooks Meadow, Hemel Hempstead, SG12 9PL UK
Copyright 2016 GV Health Ltd www.gvhealth.com

Order information

Information	Code
Cytotoxic Spill Kit	MJZ000